




INSTRUCTIONS - EXTENSIONS SRLL1, SRLL2, & SRLL3

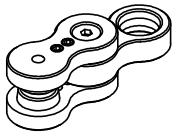

SRLL1

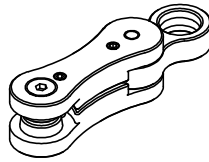


SRLL2

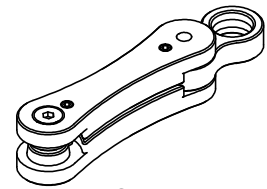

SRLL3


SRLL KITS CAN BE USED TO CHANGE THE LENGTH OR PROVIDE A CURVE TO STA-RITE LATERALS OR FOREHEAD STABILIZERS. LINKS CAN BE USED TO SHORTEN OR EXTEND LENGTHS. LINKS CAN BE COMBINED.

TYPICAL INSTALLATION - LATERAL ARM SHOWN

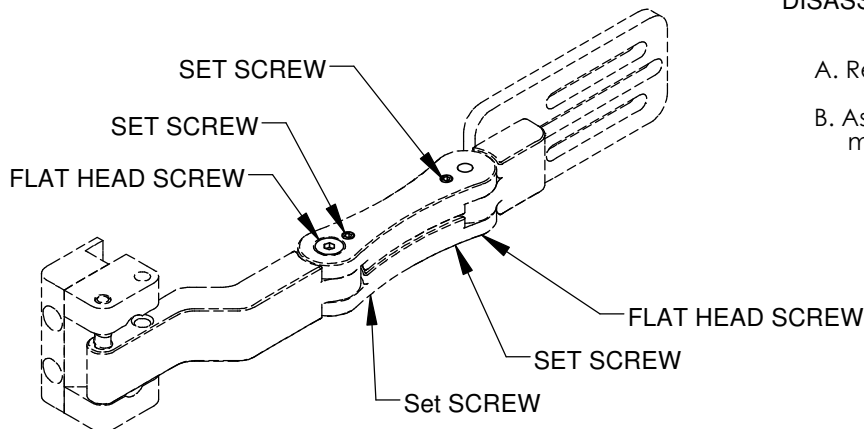

DISASSEMBLE ARM LINKS

- Remove flat head screws, 2X.
- Assembled parts may be locked together, making separation difficult. If so:
Tighten set screws to break parts apart, 4X.
Loosen set screw so head of set screw is flush with outer surface of arm.

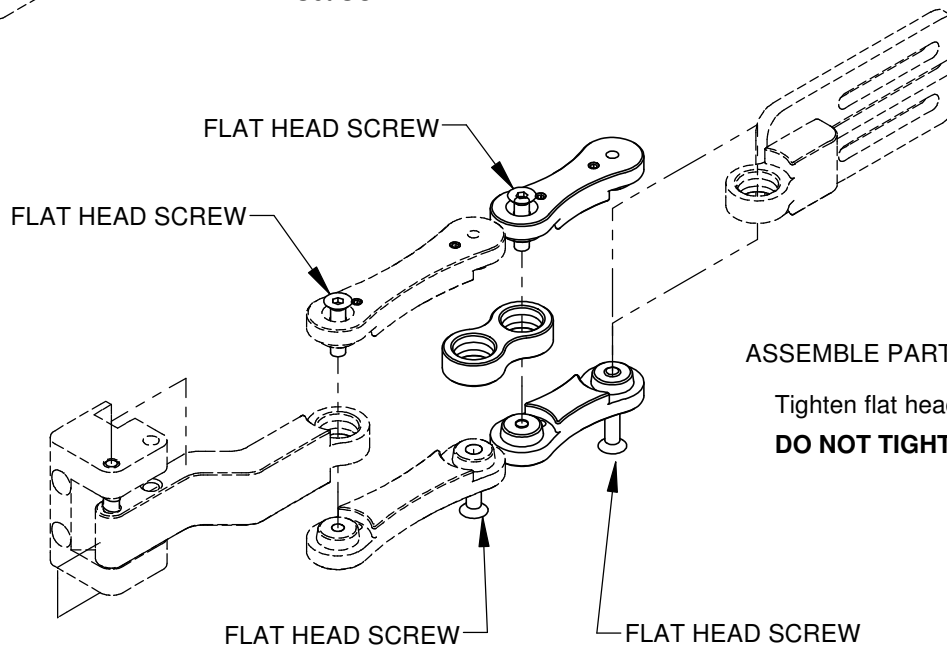

ASSEMBLE PARTS

- Tighten flat head screws, 4X.
DO NOT TIGHTEN SET SCREWS